

Tissue-Tek Genie®

anti-Granzyme B Rabbit Monoclonal Antibody [EP230]

Instructions for use

Intended use

For *in vitro* diagnostic use.

Tissue-Tek Genie® anti-Granzyme B Rabbit Monoclonal Antibody [EP230] is an antibody designed to qualitatively detect granzyme B protein in formalin-fixed, paraffin embedded (FFPE) specimen sections by immunohistochemistry (IHC) staining on the Tissue-Tek Genie® Advanced Staining System. The clinical interpretation must be made in conjunction with histological examination, relevant clinical information, other diagnostic tests and proper controls by a qualified pathologist.

Limitations

This product has been optimized for use with the default protocol for this antibody on the Tissue-Tek Genie Advanced Staining System, using Tissue-Tek Genie® reagents and FFPE specimen sections. Staining quality may diminish when used with other systems and/or reagents.

Summary and principle

Granzyme B is a serine protease expressed by cytotoxic T-lymphocytes (CTLs) and natural killer (NK) cells. It induces apoptosis in target cells of these lymphocytes. A number of studies have demonstrated that the expression of granzyme B by immunohistochemistry in several entities of extra nodal peripheral T-cell lymphoma (PTCL) and NK-cell lymphomas, including nasal and nasal-type NK/T cell lymphomas, hepatosplenic and non-hepatosplenic PTCL, enteropathy-type (ETCL) and non-ETCL

intestinal PTCL, subcutaneous panniculitis-like PTCL (SPTCL), cutaneous CD8+ epidermotropic lymphomas as well as in nodal and cutaneous CD30+ anaplastic large cell lymphomas (ALCL). The expression of cytotoxic proteins, such as granzyme B may aid in the identification and classification of extranodal T- and NK-cell lymphomas, since many of these tumors do not have specific morphology and phenotype, when used in a panel with other antibodies.

The Tissue-Tek Genie anti-Granzyme B Rabbit Monoclonal Antibody [EP230] is a primary antibody against the human granzyme B protein and is provided in buffered saline containing 1% bovine serum albumin and 0.09% sodium azide. FFPE specimen sections are placed on positively charged slides and the paraffin is removed using the Tissue-Tek Genie® Dewax Solution (REF 8865-G001), after which heat-induced epitope retrieval is performed using the Tissue-Tek Genie® High pH Antigen Retrieval Solution (REF 8744-G001).

IHC demonstration of granzyme B in FFPE specimen sections is achieved through use of the Tissue-Tek Genie anti-Granzyme B Rabbit Monoclonal Antibody [EP230] and the Tissue-Tek Genie® Pro Detection Kit, DAB (REF 8826-K250). This procedure entails the sequential application of antibody and kit components as follows:

- Tissue-Tek Genie® Protein Block
- Tissue-Tek Genie® anti-Granzyme B Rabbit Monoclonal Antibody [EP230]
- Tissue-Tek Genie® Peroxidase Block

- Tissue-Tek Genie® Link (binds to the primary antibody)
- Tissue-Tek Genie® Polymer HRP-Conjugate (binds to the link)
- Tissue-Tek Genie® DAB Substrate (visualizes the detected protein)

Tissue-Tek Genie® Hematoxylin (REF 8830-M250) is then used to visualize the nuclei of cells. The IHC stained slide is cover-slipped and the FFPE specimen section reviewed using a light microscope.

Expected results

Specificity and intended use of this antibody were validated by performing IHC staining on the Tissue-Tek Genie Advanced Staining System using FFPE normal and tumor specimen sections.

In normal tonsil, spleen, thymus, positive staining is observed in cytotoxic T-lymphocytes (CTLs) and natural killer (NK) cells. Granular cytoplasmic staining is detected on majority of tumor cells in the peripheral T-cell lymphoma.

Sensitivity and identification of granzyme B protein by this antibody may be affected by improper specimen handling. This may alter antigenicity, weaken detection and may generate false negative results.

Cellular staining pattern: cytoplasmic, granular

Positive specimen control: spleen, tonsil, selective T-cell lymphoma

Cautions and warnings

For professional use only. Take reasonable precautions when handling. Avoid contact of reagents with eyes, skin, and mucous membranes. Wear protective gloves, clothing, and eye/face protection.

Capsules filled with ready-to-use, pre-diluted, antibody are for single use only. Do not attempt to refill or add additional reagent. Discard capsule after use.

Cartridges filled with ready-to-use, pre-diluted, antibody are intended for multiple uses. Do not attempt to refill or add additional reagent. Discard cartridge when empty.

It is recommended to include appropriate controls on each specimen slide to help in identifying any deviation that might occur during the staining process.

All disposal practices must be in compliance with all Federal, State/Provincial and local laws and regulations. Refer to the SDS for further information.

Storage conditions

Store this product at 2-8°C.

Instructions for use

Tissue-Tek Genie® anti-Granzyme B Rabbit Monoclonal Antibody [EP230], capsules (REF 8234-C010):

1. Place the Tissue-Tek Genie® Reagent Dispensing Area Tag (RDA-Tag) attached to the capsule into the RDA.
2. Push the capsule into the RDA with foil side down and click the attached RDA-Tag down into place on the RDA.
3. Place the RDA on the desired station of the Tissue-Tek Genie Advanced Staining System.
4. Place the slide with the specimen section on the same station, specimen section side down.
5. Assign protocol 8234 to the same station.
6. Initiate execution of protocol 8234.
7. The RDA-Tag 8234 will be scanned and registered automatically when the staining process is initiated.
8. During the primary antibody application step, the antibody will be released from the capsule into the RDA and onto the specimen section on the slide.
9. The staining protocol continues to the end.

Tissue-Tek Genie® anti-Granzyme B Rabbit Monoclonal Antibody [EP230], cartridge (REF 8234-M250):

1. Prior to placing the cartridge on the carousel of the Tissue-Tek Genie Advanced Staining System, prime the cartridge by facing the nozzle downwards and gently pinching the nozzle tubing until the tubing is filled with the reagent.
2. Place the cartridge on the carousel.



3. Click the RDA-Tag 8234 into place on the RDA.
4. Place the RDA on the desired station of the Tissue-Tek Genie Advanced Staining System.
5. Place the slide with the specimen section on the same station, specimen section side down.
6. Assign protocol 8234 to the same station.
7. Initiate execution of protocol 8234.
8. The RDA-Tag 8234 and the cartridge will be scanned and registered automatically when the staining process is initiated.
9. During the primary antibody application step, the antibody will be dispensed from the cartridge into the RDA and onto the specimen section on the slide.
10. The staining protocol continues to the end.

Material required but not supplied

The following reagents may be required for staining but are not provided:

- Tissue-Tek Genie® Dewax Solution (REF 8865-G001)
- Tissue-Tek Genie® Wash Solution (REF 8874-G004)
- Tissue-Tek Genie® High pH Antigen Retrieval Solution (REF 8744-G001)
- Tissue-Tek Genie® Non-Immune Rabbit Ig Antibody, Negative Control (REF 8605-C010, 8605-M250)
- Tissue-Tek Genie® Pro Detection Kit, DAB (REF 8826-K250)
- Tissue-Tek Genie® Hematoxylin (REF 8830-M250)

Further information can be found on the Sakura Finetek USA website at www.sakuraus.com/Genie

Order information

Product code, product name and quantity

(REF) 8234-C010 Tissue-Tek Genie® anti-Granzyme B Rabbit Monoclonal Antibody [EP230], Ready-To-Use, 10 capsules; 1 pack.

(REF) 8234-M250 Tissue-Tek Genie® anti-Granzyme B Rabbit Monoclonal Antibody [EP230], Ready-To-Use, 250 tests, 1 cartridge; 1 unit.

NOTE: The Safety Data Sheet (SDS) is available online on the Sakura Finetek USA website at www.sakuraus.com/SDS.html

References








1. Smyth MJ, Trapani JA. Immunol Today. 1995; 16:202-206
2. Froelich CJ, et al. Immunol Today. 1998; 19:30-36
3. Kato N, et al. Am J Dermatopathol. 2003; 25:142-147
4. Balmer NN, et al. Am J Dermatopathol. 2009; 31:187-192

Contact

If located within the United States, contact Sakura Finetek USA, Inc. by calling toll free **1-800-725-8723** or contact your Sakura Finetek representative or authorized distributor.

In countries, other than the United States, contact the nearest authorized Sakura Finetek instrument distributor or representative. Contact details may be found at www.sakura.com

Symbols

	Catalog number
	Batch code
	<i>in vitro</i> diagnostic medical device
	Temperature limitation
	Use by
	Manufacturer
	Consult instructions for use
	European Conformity
	Authorized representative in the European Community

Storage: 2°C  8°C







	Sakura Finetek USA, Inc. 1750 W 214 th Street Torrance, CA 90501 U.S.A.
	Sakura Finetek Europe B.V. Flemingweg 10a 2408 AV Alphen aan den Rijn The Netherlands
Made in U.S.A.	

GS-33243 Rev. A