

Tissue-Tek Genie®

anti-BOB.1 Mouse Monoclonal Antibody [ZM74]

Instructions for use

Intended use

For *in vitro* diagnostic use.

Tissue-Tek Genie® anti-BOB.1 Mouse Monoclonal Antibody [ZM74] is an antibody designed to qualitatively detect BOB.1 protein in formalin-fixed, paraffin embedded (FFPE) specimen sections by immunohistochemistry (IHC) staining on the Tissue-Tek Genie® Advanced Staining System. The clinical interpretation must be made in conjunction with histological examination, relevant clinical information, other diagnostic tests and proper controls by a qualified pathologist.

Limitations

This product has been optimized for use with the default protocol for this antibody on the Tissue-Tek Genie Advanced Staining System, using Tissue-Tek Genie® reagents and FFPE specimen sections. Staining quality may diminish when used with other systems and/or reagents.

Summary and principle

BOB.1, B-cell Octamer Binding Protein-1, is a B-cell specific coactivator and interacts with OCT.1 and OCT.2 transcription factors. It mediates antigen-dependent germinal center formation. The expression of BOB.1 is largely to mature B-cells in germinal center, and also in mantle zone and plasma cells. The expression of BOB.1 in B-cell tumors is variable. Lymphocyte predominant (LP) cells in nodular lymphocyte predominant Hodgkin's lymphoma are BOB.1 positive. Conversely, only some cases of

classical Hodgkin's lymphoma are BOB.1 positive in Reed-Sternberg cells. In B-cell lymphomas, the highest expression levels for BOB.1 are in follicular center lymphomas, diffuse large B-cell lymphomas (DLBCL) and Burkitt lymphomas. Approximately 50% of acute myeloid leukemias express BOB.1. B-cell chronic lymphocytic leukemia (B-CLL), marginal zone lymphoma, and mantle cell lymphoma, and MALT-type score negative or display an heterogenous/weaker reactivity. Anti-BOB.1 antibody is useful for identification of B-cells in germinal centers and mantle cells, and for differentiation of lymphoma, especially for identification of the lineage of CD20 negative B-cell neoplasms and differentiation of classical Hodgkin's lymphoma (BOB.1 negative) from primary mediastinal large B-cell lymphoma (BOB.1 positive) when used in a panel with other antibodies. The strong nuclear expression of BOB.1 and OCT.2 by germinal center derived lymphomas makes these antibodies a novel class of broad-spectrum B-lineage IHC markers to aid in the differential diagnosis of lymphomas.

The Tissue-Tek Genie anti-BOB.1 Mouse Monoclonal Antibody [ZM74] is a primary antibody against the BOB.1 protein and is provided in buffered saline containing 1% bovine serum albumin and 0.09% sodium azide. FFPE specimen sections are placed on positively charged slides and the paraffin is removed using the Tissue-Tek Genie® Dewax Solution (REF 8865-G001), after which heat-induced epitope retrieval is performed using the Tissue-Tek Genie® High pH Antigen Retrieval Solution (REF 8744-G001).

IHC demonstration of BOB.1 protein in FFPE specimen sections is achieved through use of the Tissue-Tek Genie anti-BOB.1 Mouse Monoclonal Antibody [ZM74] and the Tissue-Tek Genie[®] Pro Detection Kit, DAB (REF 8826-K250). This procedure entails the sequential application of antibody and kit components as follows:

- Tissue-Tek Genie[®] Protein Block
- Tissue-Tek Genie[®] anti-BOB.1 Mouse Monoclonal Antibody [ZM74]
- Tissue-Tek Genie[®] Peroxidase Block
- Tissue-Tek Genie[®] Link (binds to the primary antibody)
- Tissue-Tek Genie[®] Polymer HRP-Conjugate (binds to the link)
- Tissue-Tek Genie[®] DAB Substrate (visualizes the detected protein)

Tissue-Tek Genie[®] Hematoxylin (REF 8830-M250) is then used to visualize the nuclei of cells. The IHC stained slide is cover-slipped and the FFPE specimen section reviewed using a light microscope.

Expected results

Specificity and intended use of this antibody were validated by performing IHC staining on the Tissue-Tek Genie Advanced Staining System using FFPE normal and tumor specimen sections.

Nuclear staining is observed in all germinal center and mantle-zone B, plasma cells, and lymphocytes in lamina propria. Nuclear staining is observed in B-cells in various tissues. Nuclei and cytoplasmic staining are observed in neoplastic cells of B-cell neoplasms, follicular center lymphoma, DLBCL, and Burkitt lymphomas. Positive staining was not observed in Reed-Sternberg cells of classical Hodgkin's lymphoma.

Sensitivity and identification of BOB.1 protein by this antibody may be affected by improper specimen handling. This may alter antigenicity, weaken detection and may generate false negative results.

Cellular staining pattern: nuclear and cytoplasmic

Positive specimen control: tonsil, DLBCL, follicular lymphoma

Cautions and warnings

For professional use only. Take reasonable precautions when handling. Avoid contact of reagents with eyes, skin, and mucous membranes. Wear protective gloves, clothing, and eye/face protection.

Capsules filled with ready-to-use, pre-diluted, antibody are for single use only. Do not attempt to refill or add additional reagent. Discard capsule after use.

Cartridges filled with ready-to-use, pre-diluted, antibody are intended for multiple uses. Do not attempt to refill or add additional reagent. Discard cartridge when empty.

It is recommended to include appropriate controls on each specimen slide to help in identifying any deviation that might occur during the staining process.

All disposal practices must be in compliance with all Federal, State/Provincial and local laws and regulations. Refer to the SDS for further information.

Storage conditions

Store this product at 2-8°C.

Instructions for use

Tissue-Tek Genie[®] anti-BOB.1 Mouse Monoclonal Antibody [ZM74], capsules (REF 8462-C010):

1. Place the Tissue-Tek Genie[®] Reagent Dispensing Area Tag (RDA-Tag) attached to the capsule into the RDA.
2. Push the capsule into the RDA with foil side down and click the attached RDA-Tag down into place on the RDA.
3. Place the RDA on the desired station of the Tissue-Tek Genie Advanced Staining System.
4. Place the slide with the specimen section on the same station, specimen section side down.
5. Assign protocol 8462 to the same station.
6. Initiate execution of protocol 8462.
7. The RDA-Tag 8462 will be scanned and registered automatically when the staining process is initiated.

8. During the primary antibody application step, the antibody will be released from the capsule into the RDA and onto the specimen section on the slide.
9. The staining protocol continues to the end.

Tissue-Tek Genie® anti-BOB.1 Mouse Monoclonal Antibody [ZM74], cartridge (REF 8462-M250):

1. Prior to placing the cartridge on the carousel of the Tissue-Tek Genie Advanced Staining System, prime the cartridge by facing the nozzle downwards and gently pinching the nozzle tubing until the tubing is filled with the reagent.
2. Place the cartridge on the carousel.
3. Click the RDA-Tag 8462 into place on the RDA.
4. Place the RDA on the desired station of the Tissue-Tek Genie Advanced Staining System.
5. Place the slide with the specimen section on the same station, specimen section side down.
6. Assign protocol 8462 to the same station.
7. Initiate execution of protocol 8462.
8. The RDA-Tag 8462 and the cartridge will be scanned and registered automatically when the staining process is initiated.
9. During the primary antibody application step, the antibody will be dispensed from the cartridge into the RDA and onto the specimen section on the slide.
10. The staining protocol continues to the end.

Material required but not supplied

The following reagents may be required for staining but are not provided:

- Tissue-Tek Genie® Dewax Solution (REF 8865-G001)
- Tissue-Tek Genie® Wash Solution (REF 8874-G004)
- Tissue-Tek Genie® High pH Antigen Retrieval Solution (REF 8744-G001)
- Tissue-Tek Genie® Non-Immune Mouse Ig Antibody, Negative Control (REF 8604-C010, 8604 -M250)
- Tissue-Tek Genie® Pro Detection Kit, DAB (REF 8826-K250)
- Tissue-Tek Genie® Hematoxylin (REF 8830-M250)

Further information can be found on the Sakura Finetek USA website at www.sakuraus.com/Genie

Order information

Product code, product name and quantity

REF 8462-C010 Tissue-Tek Genie® anti-BOB.1 Mouse Monoclonal Antibody [ZM74], Ready-To-Use, 10 capsules; 1 pack.

REF 8462-M250 Tissue-Tek Genie® anti-BOB.1 Mouse Monoclonal Antibody [ZM74], Ready-To-Use, 250 tests, 1 cartridge; 1 unit.

NOTE: The Safety Data Sheet (SDS) is available online on the Sakura Finetek USA website at www.sakuraus.com/SDS.html

References









1. Greiner A, et al. Am J Pathol. 2000; 156:501-507
2. Stein H, et al. Blood. 2001; 97:496-501
3. Saez AI, et al. Mod Pathol. 2002; 15:211-220
4. Hertel CB, et al. Oncogene. 2002; 21:4908-4920
5. Steimle-Grauer SA, et al. Virchows Arch. 2003; 442:284-293
6. Pileri SA, et al. Am J Pathol. 2003; 162:243-253
7. Browne P, et al. Am J Clin Pathol. 2003; 120:767-777
8. Garcia-Cosio M, et al. Mod Pathol. 2004, 17:1531-1538
9. Chu PG, et al. Am J Clin Pathol. 2006; 126:534-544
10. Gibson SE, et al. Am J Clin Pathol. 2006; 126:916-924
11. Hoefnagel JJ, et al. Mod Pathol. 2006; 19:1270-1276
12. McGune R, et al. Mod Pathol. 2006; 19:1010-1018
13. Hoeller S, et al. Histopathology. 2010; 56:217-228

Contact

If located within the United States, contact Sakura Finetek USA, Inc. by calling toll free **1-800-725-8723** or contact your Sakura Finetek representative or authorized distributor.

In countries, other than the United States, contact the nearest authorized Sakura Finetek instrument distributor or representative. Contact details may be found at **www.sakura.com**

Symbols

-  REF Catalog number
-  LOT Batch code
-  IVD *in vitro* diagnostic medical device
-  Temperature limitation
-  Use by
-  Manufacturer
-  Consult instructions for use
-  CE European Conformity
-  EC REP Authorized representative in the European Community

Storage: 2°C  8°C

 IVD





	Sakura Finetek USA, Inc. 1750 W 214 th Street Torrance, CA 90501 U.S.A.
	Sakura Finetek Europe B.V. Flemingweg 10a 2408 AV Alphen aan den Rijn The Netherlands
Made in U.S.A.	

GS-33257 Rev. A