

Tissue-Tek Genie®

anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10]

Instructions for use

Intended use

For *in vitro* diagnostic use.

Tissue-Tek Genie® anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10] is designed to qualitatively detect Cytokeratin LMW (CK8/18) protein in formalin-fixed, paraffin embedded (FFPE) specimen sections by immunohistochemistry (IHC) staining on the Tissue-Tek Genie® Advanced Staining System. The clinical interpretation must be made in conjunction with histological examination, relevant clinical information, other diagnostic tests and proper controls by a qualified pathologist.

Limitations

This product has been optimized for use with the default protocol for this antibody on the Tissue-Tek Genie Advanced Staining System, using Tissue-Tek Genie® reagents and FFPE specimen sections. Staining quality may diminish when used with other systems and/or reagents.

Summary and principle

Cytokeratins 8 and 18 are low molecular weight (LMW) cytokeratins and are often co-expressed and comprise the primary keratin pair in simple epithelia, such as can be found in the liver and intestine. Cytokeratin 18 is also expressed in glandular and transitional epithelial cells, and the Cytokeratin 8/18 antibody cocktail therefore labels most epithelia other than the stratified type. This antibody cocktail also labels many epithelia-derived tumor cells, and thus is useful for the identification/classification of adeno-carcinomas and

most squamous cell carcinomas when used with a panel of antibodies.

The Tissue-Tek Genie anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10] is a primary antibody against human Cytokeratin LMW (CK8/18) protein and is provided in buffered saline containing 1% bovine serum albumin and 0.09% sodium azide. FFPE specimen sections are placed on positively charged slides and the paraffin is removed using the Tissue-Tek Genie® Dewax Solution (REF 8865-G001), after which heat-induced epitope retrieval is performed using the Tissue-Tek Genie® High pH Antigen Retrieval Solution (REF 8744-G001).

IHC demonstration of Cytokeratin LMW (CK8/18) in FFPE specimen sections is achieved through use of the Tissue-Tek Genie anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10] and the Tissue-Tek Genie® Pro Detection Kit, DAB (REF 8826-K250). This procedure entails the sequential application of antibody and kit components as follows:

- Tissue-Tek Genie® Protein Block
- Tissue-Tek Genie® anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10]
- Tissue-Tek Genie® Peroxidase Block
- Tissue-Tek Genie® Link (binds to the primary antibody)
- Tissue-Tek Genie® Polymer HRP-Conjugate (binds to the link)
- Tissue-Tek Genie® DAB Substrate (visualizes the detected protein)

Tissue-Tek Genie® Hematoxylin (REF 8830-M250) is then used to visualize the nuclei of cells. The IHC stained slide is cover-slipped and the FFPE specimen section reviewed using a light microscope.

Expected results

Specificity and intended use of this antibody were validated by performing IHC staining on the Tissue-Tek Genie Advanced Staining System using FFPE normal and tumor specimen sections.

In liver, cytoplasmic staining is observed with bile duct epithelial cells, whereas hepatocytes show membranous staining. For the latter, staining may be heterogeneous, with the strongest staining in periportal zones. Kupffer cells are negative. In esophagus, cytoplasmic staining is observed in basal squamous epithelial cells, with weaker staining seen in dispersed intermediate squamous epithelial cells. Cytoplasmic staining of neoplastic cells is observed in colon adenocarcinoma, breast ductal carcinoma, renal cell carcinoma, hepatocellular carcinoma, and small cell lung carcinoma.

Sensitivity and identification of the Cytokeratin LMW (CK8/18) proteins by this antibody cocktail may be affected by improper tissue handling. This may alter antigenicity, weaken detection and may generate false negative results.

Cellular staining pattern: cytoplasmic, sometimes membrane

Positive control tissue: liver, GI tract, esophagus, and known cytokeratin LMW (CK8/18) positive tumors

Cautions and warnings

For professional use only. Take reasonable precautions when handling. Avoid contact of reagents with eyes, skin, and mucous membranes. Wear protective gloves, clothing, and eye/face protection.

Capsules filled with ready-to-use, pre-diluted, antibody are for single use only. Do not attempt to refill or add additional reagent. Discard capsule after use.

Cartridges filled with ready-to-use, pre-diluted, antibody are intended for multiple uses. Do not attempt to refill or add additional reagent. Discard cartridge when empty.

It is recommended to include appropriate controls on each specimen slide to help in identifying any deviation that might occur during the staining process.

All disposal practices must be in compliance with all Federal, State/Provincial and local laws and regulations. Refer to the SDS for further information.

Storage conditions

Store this product at 2-8°C.

Instructions for use

Tissue-Tek Genie® anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10], capsules (REF 8298-C010):

1. Place the Tissue-Tek Genie® Reagent Dispensing Area Tag (RDA-Tag) attached to the capsule into the RDA.
2. Push the capsule into the RDA with foil side down and click the attached RDA-Tag down into place on the RDA.
3. Place the RDA on the desired station of the Tissue-Tek Genie Advanced Staining System.
4. Place the slide with the specimen section on the same station, specimen section side down.
5. Assign protocol 8298 to the same station.
6. Initiate execution of protocol 8298.
7. The RDA-Tag 8298 will be scanned and registered automatically when the staining process is initiated.
8. During the primary antibody application step, the antibody will be released from the capsule into the RDA and onto the specimen section on the slide.
9. The staining protocol continues to the end.

Tissue-Tek Genie® anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10], cartridge (REF 8298-M250):

1. Prior to placing the cartridge on the carousel of the Tissue-Tek Genie Advanced Staining System, prime the cartridge by facing the nozzle downwards and gently pinching the nozzle tubing until the tubing is filled with the reagent.
2. Place the cartridge on the carousel.
3. Click the RDA-Tag 8298 into place on the RDA.



4. Place the RDA on the desired station of the Tissue-Tek Genie Advanced Staining System.
5. Place the slide with the specimen section on the same station, specimen section side down.
6. Assign protocol 8298 to the same station.
7. Initiate execution of protocol 8298.
8. The RDA-Tag 8298 and the cartridge will be scanned and registered automatically when the staining process is initiated.
9. During the primary antibody application step, the antibody will be dispensed from the cartridge into the RDA and onto the specimen section on the slide.
10. The staining protocol continues to the end.

Material required but not supplied

The following reagents may be required for staining but are not provided:

- Tissue-Tek Genie® Dewax Solution (REF 8865-G001)
- Tissue-Tek Genie® Wash Solution (REF 8874-G004)
- Tissue-Tek Genie® High pH Antigen Retrieval Solution (REF 8744-G001)
- Tissue-Tek Genie® Non-Immune Rabbit Ig Antibody, Negative Control (REF 8605-C010, 8605-M250)
- Tissue-Tek Genie® Pro Detection Kit, DAB (REF 8826-K250)
- Tissue-Tek Genie® Hematoxylin (REF 8830-M250)

Further information can be found on the Sakura Finetek USA website at www.sakuraus.com/Genie

Order information

Product code, product name and quantity

(REF) 8298-C010 Tissue-Tek Genie® anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10], Ready-To-Use, 10 capsules; 1 pack.

(REF) 8298-M250 Tissue-Tek Genie® anti-Cytokeratin LMW (CK8/18) Antibody Cocktail [EP17/DC10], Ready-To-Use, 250 tests, 1 cartridge; 1 unit.

NOTE: The Safety Data Sheet (SDS) is available online on the Sakura Finetek USA website at www.sakuraus.com/SDS.html

References










1. Wang PH. Oral Oncol. 2011; 47:775.
2. Ordonez NG. Hum Pathol. 2013; 44:1195-1215.
3. Abd El-Rehim DH, et al. J Pathol 2004; 203:661-671.
4. Oshima RG, et al. Cancer Metastasis Rev. 1996; 15:445--471.
5. Nagashio R, et al. Pathol Int. 2010; 60:71-77.

Contact

If located within the United States, contact Sakura Finetek USA, Inc. by calling toll free **1-800-725-8723** or contact your Sakura Finetek representative or authorized distributor.

In countries, other than the United States, contact the nearest authorized Sakura Finetek instrument distributor or representative. Contact details may be found at www.sakura.com

Symbols

	Catalog number
	Batch code
	<i>in vitro</i> diagnostic medical device
	Temperature limitation
	Use by
	Manufacturer
	Consult instructions for use
	European Conformity
	Authorized representative in the European Community

Storage: 2°C  8°C







	Sakura Finetek USA, Inc. 1750 W 214 th Street Torrance, CA 90501 U.S.A.
	Sakura Finetek Europe B.V. Flemingweg 10a 2408 AV Alphen aan den Rijn The Netherlands
Made in U.S.A.	

GS-32523 Rev. A